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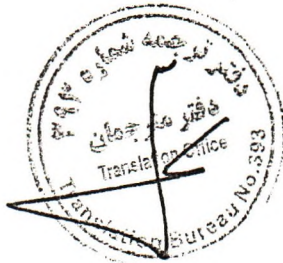
SIEMENS Healthineers

Siemens Healthcare Diagnostics Products GmbH
Emil-von-Behring-Str. 76, 35041 Marburg, Germany
siemens-healthineers.com

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Title INNOVANCE D-Dimer – potential for intermittent INNOVANCE D-Dimer Control Results Outside of Range

Date Issued March 2026

Products

Reagent	Siemens Material Number / Unique Device Identification	Kit Lot Number	Manufacturing Date	Expiration Date
INNOVANCE D-Dimer	10445980 / 00842768020117	03506, 03546, 03534, 03467	24-Mar-2025	23-Mar-2027
	10445979 / 00842768006777	03448, 03533		
	10873851 / 00630414640471	03500, 03449, 03531		
	10873850 / 00630414640235	03530, 03451		

Issue Description

Siemens Healthineers has confirmed through the investigation of customer complaints and internal investigation the potential for intermittent invalid quality control (QC) results for INNOVANCE D-Dimer component lot 56xx60 (contained in the above-mentioned Kit Lot Numbers).

If QC results are not within the acceptable range initially perform repeat testing in accordance with the Quality Control section of the Instructions for Use.

When the INNOVANCE D-Dimer Control results are within the assigned range according to the Table of Assigned Value (TAV), patient results are not impacted and are valid for reporting.

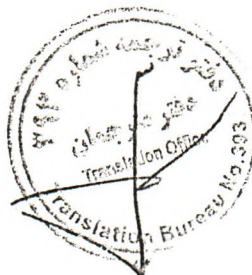
The issue is isolated to INNOVANCE D-Dimer component lot 56xx60.

Siemens Healthineers is actively investigating the root cause of this issue.

Customer Actions

For the product(s) listed in the table above, please perform the following steps:

- If QC results are within the acceptable range patient results can be reported.
- If QC results are not within the acceptable range and cannot be achieved upon repeat testing, in accordance with the Quality Control section of the Instructions for Use, complete and return the Product Replacement Form attached to this letter to request no-charge replacement product from your local Siemens Healthineers or distributor office.

Please review your inventory of these products and assess your laboratory's replacement needs.

- Please retain this letter with your laboratory records and forward this letter to those who may have received the product.

Single Registration
Number (SRN)

DE-MF-00005039

We apologize for the inconvenience this situation may cause. If you have any questions, please contact your Siemens Healthineers Customer Care Center or your local Siemens Healthineers technical support representative.

Sincerely yours,

This letter was created electronically and is valid without signature.

I.V. Nils Neumann

Director Quality Systems & Compliance

I.V. Thomas Scholz

Marketing Manager Global Marketing

All trademarks are the property of their respective owners.

Siemens Healthineers

Siemens Healthcare Diagnostics Products GmbH

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